


# QuoroSkin

Natural Bovine Collagen Wet Sheet

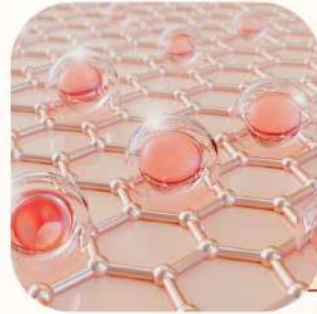

NATURAL 100% PURE  
COLLAGEN WET SHEET

AN  
INTELLIGENT  
WOUND  
DRESSING



INDICATIONS	<ul style="list-style-type: none"><li>• 1st &amp; 2nd Degree non-infected burn wounds</li><li>• Pressure injuries, diabetic ulcers, venous ulcers</li><li>• Ulcers caused by mixed vascular aetiologies</li><li>• Donor and graft sites</li><li>• Abrasions</li></ul>
ADVANTAGES	<ul style="list-style-type: none"><li>• Releases 100% pure collagen to deactivate &amp; down-regulate excess MMPs</li><li>• Delays biofilm formation &amp; reduces bio-burden</li><li>• Selectively permeable with high Bacterial Filtration Efficiency (BFE)</li><li>• Accelerates all 4 stages of wound healing - haemostatis, inflammation, proliferation &amp; remodelling</li><li>• Facilitates angiogenesis &amp; re-epithelialization</li><li>• Quick &amp; firm adherence to wound surface</li><li>• Reduces trauma of removal - ideal for both adult &amp; paediatric patients*</li></ul>
PRECAUTIONS	Do not use if the pack is damaged or open
CONTRA-INDICATIONS	Avoid using QuoroSkin on patients who are hypersensitive to collagen
SIZES	5cmx5cm, 10cmx10cm, 10cmx15cm, 15cmx20cm

#### Application Procedure:

- Cleanse & decontaminate the wound area thoroughly
- Debride infected or necrotic wounds thoroughly
- Ensure satisfactory haemostasis on skin donor site
- Remove QuoroSkin wet sheet gently & rinse thoroughly in sterile saline to remove the preserving medium
- Apply firmly on raw area without leaving any air bubbles
- Cover the wound with an absorbent protective dressing like QuoroFoam, QuoroFix-N, QuoroFix-T if required

**Quor***t*  
WOUND CARE  
THE WOUND MANAGEMENT EXPERTS

www.quorit.com  
www.docuses.com  
sales@docuses.com  
+91-93138 07696




# QuoroSkin

## Natural Bovine Collagen Wet Sheet

- ✓ RELEASES 100% PURE COLLAGEN TO DEACTIVATE & DOWN-REGULATE EXCESS MMPs
- ✓ DELAYS BIOFILM FORMATION & REDUCES BIO-BURDEN
- ✓ SELECTIVELY PERMEABLE WITH HIGH BACTERIAL FILTRATION EFFICIENCY (BFE)
- ✓ ACCELERATES ALL 4 STAGES OF WOUND HEALING - HAEMOSTASIS, INFLAMMATION, PROLIFERATION & REMODELLING
- ✓ REDUCES TRAUMA OF REMOVAL - IDEAL FOR BOTH ADULT & PAEDIATRIC PATIENTS\*

- » 1st & 2nd Degree Burns
- » Pressure Injuries, Diabetic or Venous Ulcers
- » Ulcers Caused by Mixed Vascular Aetiologies
- » Donor & Graft Sites
- » Chronic Wounds
- » Available in 5x5, 10x10, 10x15 & 15x20 cm

**AN  
INTELLIGENT  
WOUND  
DRESSING**

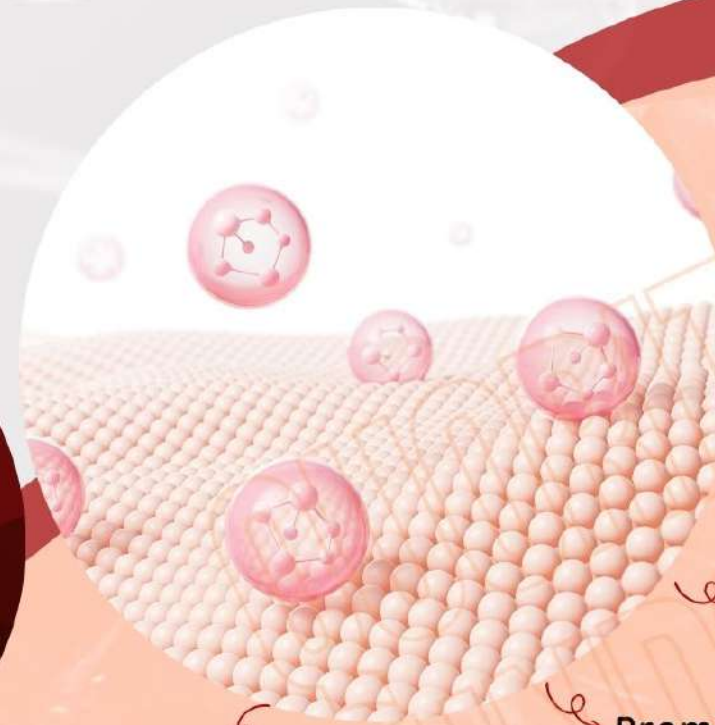

Protection from  
Bacteria & External  
Contaminants

Modulates  
Inflammatory  
Response

Facilitates  
Angiogenesis

Promotes  
Re-Epithelialization

Heals with Minimal  
or No Scarring

**Quorit**  
WOUND CARE

Wound Management Experts

[WWW.QUORIT.COM](http://WWW.QUORIT.COM)

\*(International Surgery Journal Gupta M et al. Int Surg J. 2021 Nov;8(11):3339-3343 <http://www.ijurgery.com>)