


QuoroSkin-D

Natural Bovine Collagen Dry Sheet

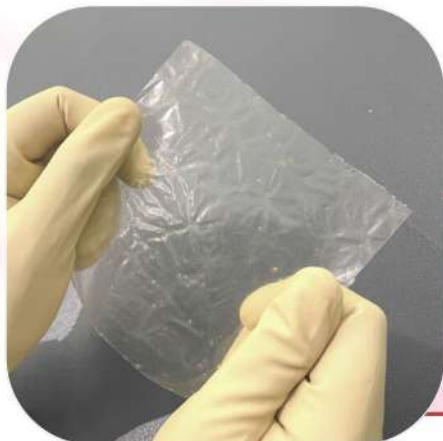

RECONSTITUTED PURE
BOVINE COLLAGEN DRY
SHEET

AN
INTELLIGENT
WOUND
DRESSING



A preferred biological dressing for its innate characteristics of non-inflammatory, excellent breathability & non-permeability to bacteria

| | |
|--------------------|---|
| INDICATIONS | <ul style="list-style-type: none">• 1st & 2nd Degree non-infected burn wounds• Pressure injuries, diabetic ulcers, venous ulcers• Ulcers caused by mixed vascular aetiologies• Donor and graft sites• Abrasions• Temporary dressing after escharotomy/tangential excision in 3rd degree burn |
| ADVANTAGES | <ul style="list-style-type: none">• Accelerates all 4 stages of wound healing - haemostatis, inflammation, proliferation & remodelling• Facilitates angiogenesis & re-epithelialization• Ultra-thin dry sheet facilitates easy application and conformability• Dissolves into the wound, requires no removal after healing• Increases degree of contraction & accelerates wound closure• Excellent patient compliance as high as 96.7% * |
| PRECAUTIONS | <ul style="list-style-type: none">• Do not use if the pack is damaged or open• Avoid in grossly infected wounds |
| CONTRA-INDICATIONS | Avoid using QuoroSkin-D on patients who are hypersensitive to collagen |
| SIZES | 2cmx2cm, 5cmx5cm, 10cmx10cm, 10cmx15cm, 15cmx20cm |

Application Procedure:

- Cleanse & decontaminate the wound area thoroughly
- Debride infected or necrotic wounds thoroughly
- Ensure satisfactory haemostasis on skin donor site
- Apply QuoroSkin-D on raw area without leaving any air bubbles
- Avoid over-stretching the sheet
- Cover the wound with an absorbent protective dressing like QuoroFoam, QuoroFix-N, QuoroFix-T if required
- Dry Sheet peels off as epithelialization occurs - do not forcibly remove

Quor
WOUND CARE

THE WOUND MANAGEMENT EXPERTS

www.quorit.com

www.docuses.com

sales@docuses.com

+91-93138 07696



QuoroSkin-D

Reconstituted Pure Bovine Collagen Sheet

- ✓ ACCELERATES ALL 4 STAGES OF WOUND HEALING - HAEMOSTATIS, INFLAMMATION, PROLIFERATION & REMODELLING
- ✓ ULTRA-THIN DRY SHEET FACILITATES EASY APPLICATION AND CONFORMABILITY
- ✓ DISSOLVES INTO THE WOUND, REQUIRES NO REMOVAL AFTER HEALING
- ✓ INCREASES DEGREE OF CONTRACTION & ACCELERATES WOUND CLOSURE
- ✓ EXCELLENT PATIENT COMPLIANCE AS HIGH AS 96.7% *

AN INTELLIGENT WOUND DRESSING

- › 1st & 2nd Degree Non-Infected Burn Wounds
- › Temporary Dressing After Escharotomy/Tangential Excision in 3rd Degree Burn
- › Pressure Injuries, Diabetic & Venous Ulcers
- › Ulcers Caused by Mixed Vascular Aetiologies
- › Donor & Graft Sites
- › Available in 5x5, 10x10, 10x15 & 15x20 cm



Highly Conforming to Patient's Body

Aids in Uptake & Bio-availability of Fibronectin

Accelerates Wound Closure & Healing

Heals with Minimal or No Scarring

No Trauma of Removal - Dissolves into Wound

Quor/t
WOUND CARE

Wound Management Experts

WWW.QUORIT.COM

*(International journal of contemporary medical research| Volume 5 | Issue 10 | October 2018)